STAT/S[®]

1.1, 1.1L, 4.1, 6.1 - Contra-angles

1.1, 1.1L, 4.1, 6.1 - Contre-angles

Winkelstücke 1.1, 1.1L, 4.1, 6.1

1.1, 1.1L, 4.1, 6.1 - Contrangoli

1.1, 1.1L, 4.1, 6.1 - Contra-ángulos

Operating instructions

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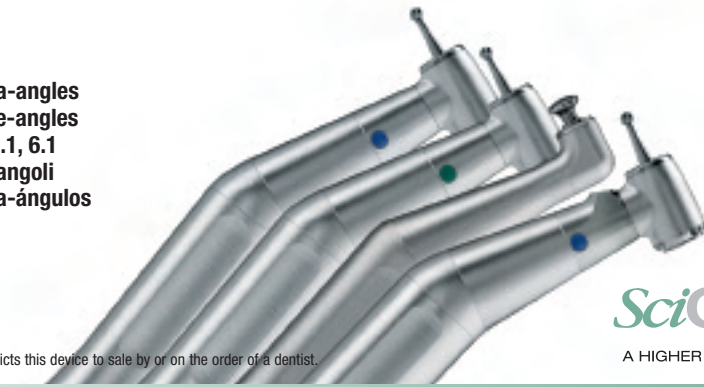
Manual del Operador

Caution: In the U.S.A., Federal Law restricts this device to sale by or on the order of a dentist.

SciCan

A HIGHER STANDARD

Distributed by:

SciCan Ltd. 1440 Don Mills Road, Toronto ON M3B 3P9 CANADA

www.scican.com

Phone: +1 416 445 1600 Toll free: 1-800-667-7733 Fax: +1 416 445 2727

Manufactured by:

SciCan GmbH

Kurzes Geländ 10
D-86156 Augsburg
GERMANY

Phone: +49 (0) 821 56 74 56-0

Fax: +49 (0) 821 56 74 56-99

SciCan Medtech

Alpenstrasse 16
6300 Zug
SWITZERLAND

Phone: +41 (0) 41 727 7027

Fax: +41 (0) 41 727 7029

SciCan Inc.

701 Technology Drive
Canonsburg, PA 15317
USA

Phone: +1 724 820 1600

Fax: +1 724 820 1479

Toll free: 1-888-632-6031

Introduction

Dental contra-angles, with or without light, push-button chuck, 3 internal water spray. Use in dentistry for general dentistry work.

Important Information

These instructions, STAT/S™ contra-angles correspond to the product described in the heading.

The device must not be started without a bur inserted into the chuck. To avoid overheating of the handpiece, which could cause burns, the push button auto chuck should not be pressed inadvertently while the instrument is rotating. Soft tissues (tongue, cheeks, lips etc.) must be protected by deflection with a retractor or other dental instrument.

Never mount an instrument on a rotating motor. To ensure the STAT/S contra-angle functions with maximum efficiency, it must be cooled by an air supply of 8-10 norm liter/min on the nose of the micromotor.

Application

The STAT/S contra-angle is only to be used by a qualified dental practitioner, in conjunction with the provision of dental treatment to dental patients. Any other use may be dangerous and is strictly forbidden. The STAT/S contra-angle is a medical product according to the applicable national provisions of law.

Information

The technical specifications, illustrations and dimensions contained in these instructions are given only as a guide. They may not be the subject of any claim. The manufacturer reserves the right to make technical improvements to its equipment, without amending these instructions. For all additional information, please contact SciCan at the address indicated on overleaf.

Generalities

The device must be used by a competent person in accordance with the current legal provisions concerning industrial safety, health and accident prevention regulations, and these working instructions. In accordance with these requirements, the operator:

- must only use working devices that are in perfect working order; in the event of irregular functioning, excessive vibration, abnormal heating or other signs indicating malfunction of the device, the work must be stopped immediately; in this case, contact a repair center that is approved by SciCan;

- must ensure that the device is used only for the purpose for which it is intended, must protect himself, his patients and third parties from any danger, and must avoid contamination through the use of the product.

The device is intended for medical treatment only; any use other than that for which this product is intended is not allowed and may prove to be dangerous. The medical device meets all the current national legal requirements. Excess material from products used for maintenance (lubricants, cleaning products and disinfectants) may penetrate into the electric brush motor and interfere with its functioning. It is essential to follow the maintenance instructions accompanying each product.

Recommendations

Adhere to the Instructions for Use, in accordance with the instructions of the bur manufacturer. Never use a bur with an incorrect shaft, as there is a risk of this becoming detached during the treatment and causing injury to you yourself, to your patient and to third parties. The only burs to be used are those corresponding to the ISO 1797 norm. They should be made from steel or hardened metal. Rest the device on a suitable support in order to eliminate the risk of injury to you yourself, to your patient and to third parties.

Guarantee

The manufacturer grants the operator a warranty covering all functional defects, material or production faults. The device is covered by this warranty for 2 years from the date of invoicing. In the event of justified claims, the manufacturer or its authorized representative will fulfill the company's obligations under this guarantee by repairing or replacing the product free of charge. Any other claims, of whatever nature, in particular in the form of a claim for damages and interest, are excluded. The manufacturer shall not be held responsible for damage or injury and the consequences thereof, resulting from:

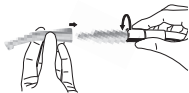
- excessive wear and tear
- improper handling
- non-observance of the instructions for installation, operation and maintenance
- unusual chemical, electrical or electrolytic influences
- poor connections, whether of the air water or electricity supply. The guarantee does not cover flexible fiber optic type conductors, or any parts made of synthetic materials.

The guarantee shall become null and void if the damage and its consequences are due to improper manipulation of the product, or modifications to the product carried out by persons not authorized by the manufacturer. Claims under the terms of the guarantee will be considered only on presentation, together with the product, of the invoice or the consignment note, on which the date of purchase, the product reference and the Serial No. should be clearly indicated.

Installation

The STAT/S contra-angle should be sterilized prior to being operated for the first time. Do not connect or disconnect the handpiece while the motor is running. Doing so could result in damage to the handpiece and motor.

Connection:

Push the contra-angle onto the E-Type coupling of the motor and twist until the guide stud locks into place. Before each use, check the connection by pulling the contra-angle to be sure it is securely attached.

Disconnection:

Pull off the contra-angle from the E-Type coupling in an axial direction.

Inserting the Bur:

Insert the press button firmly with the thumb. At the same time, insert the bur into the chuck as far as it will go. Exert traction on the bur to check that it is correctly in place. Always wear gloves or other adequate protection when checking and using the device.

Removing the Bur:

After the bur stops rotating, press the push button firmly and remove the bur. Do not press the push button while the bur and chuck are in motion. This may result in damage to the chuck and possible injury.

Maintenance

Dental handpieces should be cleaned / disinfected, lubricated and sterilized before the first use and between each patient use¹.

Only use maintenance products and/or components approved by SciCan such as SciCan STATCARE™ spray or the SciCan automatic lubrication device STATMATIC™. The use of other products and components can void the warranty. The device is supplied in non-sterile form. Before use, please follow the instructions under this heading. Check that the sterilizer and the water that is used are clean. After each sterilization cycle, remove the device from the sterilizing apparatus immediately, in order to reduce the risk of corrosion. The Statim sterilizer is recommended for maximum longevity.

1) may vary by country regulations.

Recommended reprocessing methods according to DIN EN ISO 17664

The operator should be aware of personal health risks when reprocessing contaminated dental handpieces and should wear gloves and eye protection.

- Wipe off any debris or blood contamination at chair side.
- Do not immerse dental handpieces in disinfection solutions.
- Reprocess immediately after use.
- Remove any rotary instruments (burs, files, etc.) before reprocessing.

Cleaning and Disinfection

Please note that STATIS dental handpieces should not be cleaned in an ultrasonic bath.

Attention: Do not use products containing acetone, chlorine or bleach as disinfectants

Manual External Cleaning and Disinfection

Cleaning:

After each use, the device must be thoroughly scrubbed either under water (potable water) or with 60 - 70% alcohol solution.

Disinfection

Do not use products containing acetone, chlorine or bleach as disinfectants. When applying disinfection sprays/wipes, follow the instructions for use of these materials.

FOR GERMAN USERS:

In order to efficiently reprocess dental handpieces, internal cleaning and disinfection can be achieved through the use of ALPRO WL-clean*, ALPRO WL-cid* and ALPRO WL-dry*.

*ALPRO WL-clean, ALPRO WL-cid and ALPRO WL-dry are registered trademarks of ALPRO MEDICAL GMBH, Mooswiesenstrasse 9, 78112 St. Georgen, Germany.

Mechanical External and Internal Cleaning and Disinfection

In Europe, a validated washer/disinfector such as Hydrim C51wd or M2 is recommended. In all cases the instruments need to be thoroughly dried after the automated washing process.

Be aware of the possible risks from the moist environment of the thermal cleaning and disinfection unit, when leaving the instruments for extended periods in the chamber with an unopened door.

Drying

Manual Drying

Apply dry air to the internal and external surfaces until surface water is no longer visible.

Mechanical Drying

Normally thermal washer/disinfectors have an automatic drying programme included. Units with active drying for internal lumens are recommended.

Lubrication

After cleaning and/or disinfecting the STATIS handpieces it is important to maintain/lubricate the handpieces with SciCan Statcare Spray or use an automated care system such as the Statmatic maintenance unit. Lubricate each STATIS Instrument after the cleaning and disinfection process prior to sterilisation. For those countries where sterilisation or thermal cleaning & disinfection are not prescribed, lubricate at least twice a day. SciCan only recommends the use of SciCan STATCARE Spray or STATMATIC maintenance unit.

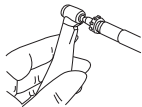
Manual Lubrication

- Remove rotary instruments from instrument chuck before cleaning, disinfection and lubrication.
- Place the instrument in a cloth or paper towel so as to catch any debris and foreign matter expelled by the atomizer jet.
- Insert the nozzle of the lubricant into the back of the instrument sleeve.
- Spray the STATCARE Spray for 1-2 seconds.
- Run the handpiece for 15-20 seconds to remove any residual debris and oil.

Manual Care of the Chuck:

It is recommended that the chuck system be cleaned and lubricated at least once a week. After removing the rotary instrument, use the tip of the nozzle of the STATCARE spray can to spray into the opening of the chuck. Thereafter, please follow the care and maintenance instructions provided by the manufacturer.

Mechanical Lubrication:

- Remove rotary instruments from the instrument chuck prior to lubrication.
- We recommend using the SciCan STATMATIC maintenance unit. Place up to 3 handpieces in the device for time efficient lubrication (15 sec./handpiece) and automatic optimized maintenance.

Automated Care of the Chuck

It is recommended that the chuck system be cleaned and lubricated at least once a week. After removing the rotary instrument, but before putting the instrument in the STATMATIC lubrication unit, use the integrated tip-adapter for cleaning and lubrication of the chuck mechanism.

Wrapping for Sterilisation

SciCan recommends the sterilisation of STATIS dental handpieces and instruments in the STATIM cassette autoclave. Where local regulations require the wrapping of dental handpieces for subsequent storage, it is important that the sterilisation bags are large enough to incorporate the instrument in such a way that the bag is in no way pressurized. The sterilisation bag must be compliant with local hygiene standards and must be designed for the use in the sterilisation process.

Sterilisation of dental handpieces

Process: STATIS devices can be sterilised in steam autoclave units following local guidelines and autoclave manufacturer's instructions.

Recommended sterilisation methods/parameters for unwrapped and wrapped loads are as follows:

According to American and Canadian standards:

Pressure pulse: 134°C - min. sterilisation holding time 3.5-6 minutes

Pre-vacuum: 134°C - min. sterilisation holding time 3.5-10 minutes

Gravity: 134°C - min. sterilisation holding time 15 minutes

According to EN13060:

For dental handpiece sterilisation a B-cycle or validated S-cycle process should be chosen.

Please note that in case of sterilisation in an S-cycle, the manufacturer of the steriliser has to specify whether dental handpieces can be sterilised in their S-cycle

Attention: Lubricate the STATIS handpieces before sterilisation! STATIS handpieces can withstand a temperature of up to 138°C.

Be aware of the possible risks of the potentially moist environment of the steriliser when leaving the instruments for extended periods in the chamber with an unopened door.

Storage

Reprocessed products should be stored wrapped in a dust-free, dry, dark environment at room temperature. Please be aware that wrapped sterile instruments have a limited shelf life (observe local guidelines).

Light

The STAT/S 1:1L is fitted with a double glass bar that is resistant to sterilization.

Environment

Temperature between -40°C (-40°F) and 70°C (158°F), relative humidity between 10% and 100%, atmospheric pressure 50 kPa to 106 kPa (7,3 to 15,3 psi).

Servicing

Never disassemble the device. For all modifications and repairs, we recommend that you contact your regular supplier or SciCan directly. The manufacturer recommends inspection of the instruments on a regular basis.

Specifications

STAT/S 1.1 L, STAT/S 1.1

Drive Speed: Maximum of 40,000 rpm

Identification: Single blue dot

Transmission: 1:1

Maximum Speed: 40,000 RPM

STAT/S 4.1

Drive Speed: Maximum of 40,000 rpm

Identification: Single green dot

Transmission: 4:1

Maximum Speed: 10,000 RPM

STAT/S 6.1 P

Drive Speed: Maximum of 40,000 rpm

Transmission: 6:1

Maximum Speed: 6,666 RPM

Classification

Class IIA in accordance with European Directive 93/42/EEC
and in accordance with medical directives.

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STAT/S, STATCARE, and STATMATIC are trademarks of SciCan Ltd. OPTIM, HYDR/M and STAT/M are a registered trademark of SciCan Ltd.
STAT/S , STATCARE et STATMATIC sont des marques de fabrique de SciCan Ltd. OPTIM est une marque déposée de SciCan Ltd.
STAT/S, STATCARE und STATMATIC sind Marken von SciCan Ltd. OPTIM ist eine eingetragene Marke von SciCan Ltd.